



3-15. Title Dubbing

When the DVO-100MD records data on a DVD disc, the same data is recorded and retained in the HDD. The titles recorded in the HDD can be dubbed to another DVD disc in the following steps. For how to operate the menu, refer to Section 5-2.


1. Press the OPEN/CLOSE button to open disc tray and insert a disc.
2. Press the ← button and MENU button simultaneously to enter the MAINTENANCE menu.

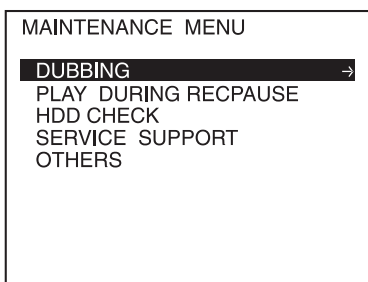

3. Press the → button to enter the DUBBING menu.

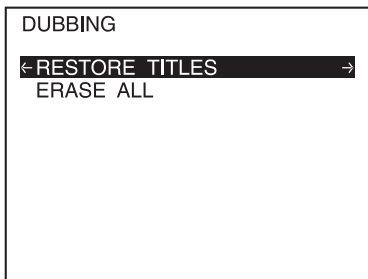


4. Press the → button to enter the RESTORE TITLES menu.

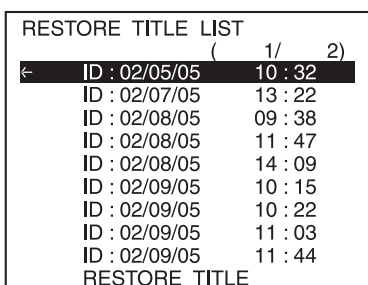


The title names are displayed in the recorded month/date/year/hour/minute/second by default.


5. Use the ↓ button to highlight the title you want to dub, and press the SET button to select. An asterisk (*) appears next to the selected titles. Press the SET button again to delete the selection. The NEXT/DOWN and PREV/UP buttons scroll between multiple pages of titles. If the list of titles continues for several pages, use the NEXT/DOWN and PREV/UP buttons to scroll the pages. The titles are dubbed to the DVD disc in the selected order.

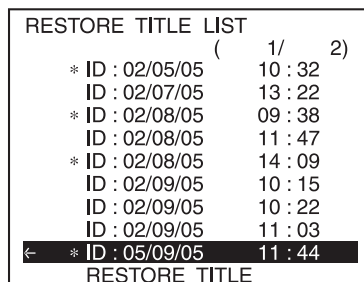


6. After selecting the titles, select RESTORE TITLE and press the → button.

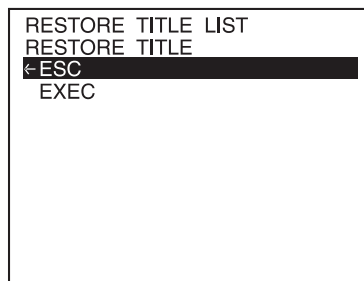

7. Select EXEC with the ↓ button and press the SET button. The confirmation message “READY?” appears.

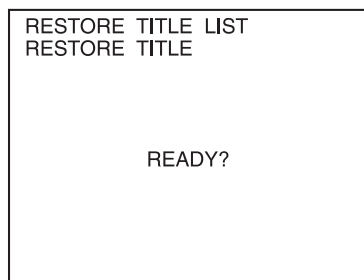

8. Press the SET button again. The remaining time appears during dubbing as shown below. “COMPLETE!” appears when finished.

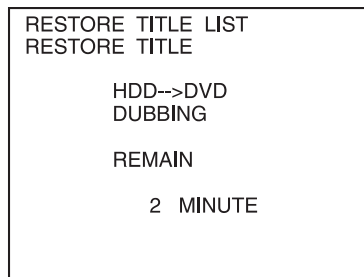

Note

The following error message appears if there is not sufficient disc space. Use another disc with sufficient unused space, or check the countermeasures noted below.

Messages during dubbing

Message		Meaning/countermeasures
Front panel display	Monitor screen	
title FULL!	TOO MANY TITLES.	Number of titles exceeds the limit (49 max.) Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.
index FULL!	TOO MANY INDEXES.	Number of indexes exceeds the limit (254 max.) Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.
DISC FULL!	DISC REMAIN AREA IS NOT ENOUGH.	Insufficient free space on the disc. Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.