




Sony DVD Recorder Model DVO-1000MD
Dubbing of HDD Files (Retrieve Studies from the HDD)





When The DVO-1000MD Records Data to DVD disk, the same Data is Recorded and Retained to the HDD


Patient Studies (Titles) can be Retrieved from HDD and Recorded to DVD+RW


Procedure


- 1) Press the OPEN/CLOSE button to open the Disc Tray and insert a DVD+RW Disc
- 2) Press the  button and **MENU** button simultaneously to enter the MAINTENANCE menu

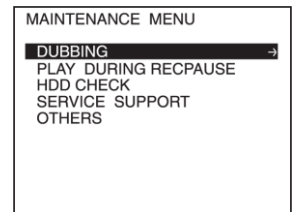

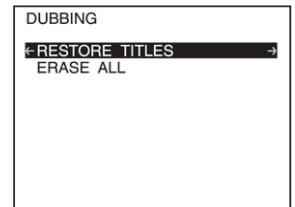


- 3) Press the  button to enter the DUBBING menu

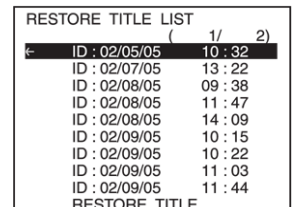


- 4) Press the  button to enter the RESTORE TITLES menu
Title names are displayed in the MM/DD/YY format




- 5) Use  and  buttons to highlight the title(s) you wish to restore. Press the **SET** button to select, an asterisk (*) will appear next to each selected title. To deselect a title press the **SET** button again

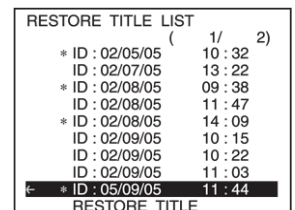


Titles are restored to DVD+RW in the order they are selected


- 6) After selecting titles, select RESTORE TITLE and press the  button

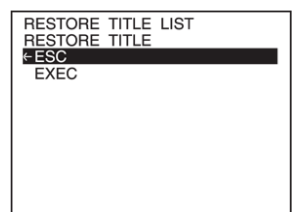

- 7) Select **EXEC** and press the **SET** button. The confirmation "READY?" appears

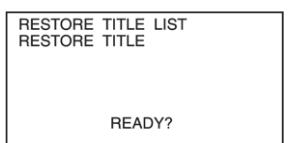

- 8) Press the SET button again. The remaining time appears during dubbing as shown. "COMPLETE" appears when finished

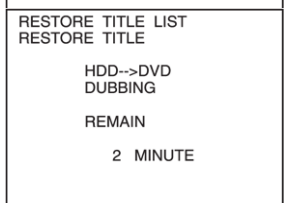

Note

The following error message appears if there is not sufficient disc space. Use another disc with sufficient unused space, or check the countermeasures noted below.

Messages during dubbing

Message		Meaning/countermeasures
Front panel display	Monitor screen	
title FULL!	TOO MANY TITLES.	Number of titles exceeds the limit (49 max.) Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.
index FULL!	TOO MANY INDEXES.	Number of indexes exceeds the limit (254 max.) Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.
DISC FULL!	DISC REMAIN AREA IS NOT ENOUGH.	Insufficient free space on the disc. Remedy: Delete titles not needed on the disc, or reselect the titles used during dubbing.