



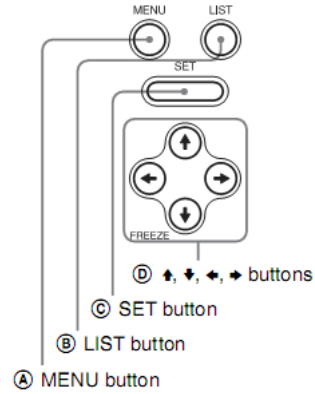

Medical Limited

**Sony DVD Recorder Model DVO-1000MD
Deletion of HDD Files**

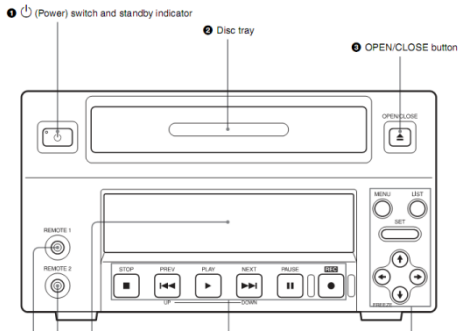

Action	Display
Switch on Switch to Rear Panel Plus On Button to Front Panel	'Welcome' followed by 'NO DISC' (If display is '0101' or similar, remove DVD Disc)
Press and Hold 'Left' Arrow Key Press 'Menu' Key Release both Keys	'Disc Copy'
Press 'Down' Arrow Key	'Dubbing'
Press 'Right' Arrow Key	'Restore Titles'
Press 'Down' Key	'Erase All'
Press 'Right' Key	'ESC'
Press 'Down' Key	'EXEC'
Press 'Set' Key	'Ready'
Press 'Set' Key	'Erasing' followed by 'Disc Copy'
Press 'Menu' Key	'NO DISC'
Switch Off	'Good Bye'