



MAS SD Video Recording System - Quick Start Guide

Connect Mains Power and Confirm:

Sony Recorder is on
VCS-2 Lite is on
Sony Monitor is on
Phono Pre Amp is on (Sound Systems)
Select VCS-2 Lite 'Fluoro' or 'Acq' (Default to 'Fluoro')
X-Ray Image is displayed

Input Patient Information (Optional):

Select HVO-550MD *Menu*
Select *Recording Preparation*
Input Data either by the 'On Screen' or 'Attached' Keyboard
(See 'Use of Keyboard', within our website 'Downloads and Support')
Select *Done*

Recording Test:

Activation from X-Ray: Recording Start
Deactivation from X-Ray: Recording Pause
Select HVO-550MD *Stop*: Recording End

Save Recording to Multimedia (DVD, Optical Flash Disk):

Select *Menu*

Navigate to *Recording List*

Enter

Navigate to *Play*

Enter

Navigate to *Save To*

Check Recordings for Save

Navigate to *Execute*

Enter

Select Relevant Recording Media

Insert Relevant Media

Select *Execute*

Select *Close*

Playback from HDD:

Navigate to *Recording List*

Enter

Highlight recording to *Playback*

Select *Play*

Playback from Multimedia:

Recordings are replayed from PC or Laptop

Note - HVO-550MD does not have the functionality to replay from Multimedia