



Medical Limited

**Sony Video Recorder Model HVO-550MD
Simultaneous Recording to HDD and DVD**

Action		Display
1	Select 'Menu' Button from Recorder Panel	Highlights 'Recording Preparation'
2	Scroll to 'Settings'	Highlights 'Settings'
3	Select 'Enter' (From Keyboard)	Highlights 'User Settings'
4	Scroll to 'System Admin Settings'	Highlights 'System Admin Settings'
5	Select 'Enter'	Highlights 'OK'
6	Select 'Enter'	Highlights 'Language'
7	Scroll to 'Initial User Settings'	Highlights 'Initial User Settings'
8	Select 'Enter'	Highlights 'Quality'
9	Scroll to 'Save'	Highlights 'Save'
10	Scroll to 'External Media'	
11	Select 'Enter'	
12	Scroll to 'DVD'	Highlights 'DVD'
13	Select 'Enter'	Highlights 'DVD'
14	Scroll to 'Record Save To Media'	
15	Select 'Enter'	
16	Scroll to 'DVD'	Highlights 'DVD'
17	Select 'Enter'	Highlights 'DVD'
18	Scroll to 'Apply'	Highlights 'Apply'
19	Select 'Enter'	Highlights 'Language'
20	Select 'Esc'	Highlights 'User Settings'
21	Select 'Esc'	Highlights 'Recording Preparation'
22	Select 'Esc'	

Notes:

With the above configuration, HVO-550MD records simultaneously to both HDD and DVD.
HVO-550MD will not record simultaneously to DVD and USB.

To disable simultaneous recording, in step 12, scroll to 'Not Specified'