



Medical Limited

**Sony Video Recorder Model HVO-550MD
Use of Keyboard**

Action	Display
Select 'Menu' Button from Recorder Panel	Menu (Highlights 'Recording Preparation')
Select 'Enter' (from Keyboard)	Recording Preparation (Highlights 'Patient ID')
Select 'Enter'	Patient ID (Highlights 'Patient ID Entry Field')
Populate Entry Field from Keyboard	(As Typed)
Select 'Enter'	Recording Preparation (Highlights 'Patient ID' as Typed)
Scroll to other Patient Detail Fields	
Populate Fields as above	
Scroll to 'Done'	Highlights 'Done'
Select 'Enter'	
Recorder is 'Prepared for Recording'	
Note: Number of digits in Patient ID can be set between 4 and 16 Default is 8 Setting is within System Admin Settings / Function Settings / General 1	